

PACS Print® Software setup and configuration:

1 Start PACS Print Software.

The first time the program is opened the configuration screen will appear. Fill in your hospital's or institution's name (optional). The Default Printer setting indicates the default Windows® printer established for the computer. For the DICOM Parameters setting, enter the AETitle. Then, if desired, check PACS Print to start automatically when the computer is booted up. For the Port Number, the standard default value for medical devices is Port 104.

2 Configure your printer.

From Printer Preferences, select the appropriate printer and configure for the desired paper size and type. Configure PACS Print as a DICOM print option from modalities and workstations. Follow manufacturer's instructions for setting up a DICOM printer. Use the IP address for the host PC, AETitle and Port Number as required. PACS Print can be used as either a color or grayscale DICOM printer.

3 Refer to the complete PACS Print Software manual for additional information.

4 Start Printing.

For lifetime activation of PACS Print, call our toll-free number: 1-866-342-6629. Record the following numbers for future reference.

_____ Serial Number (located on disk label)
_____ PC ID Number (obtained from the PACS Print start-up screen)
_____ Build Number (obtained from the PACS Print start-up screen)
_____ Activation Key (provided by NAI at time of activation)

The Serial Number, PC ID, Build Number and Activation Key are required to activate the Software.